


## MATERIALS

# FAST FACE SHIELD

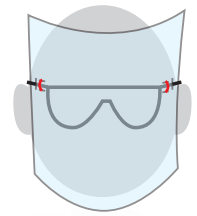

### x1 pair of glasses

- Wearer's prescription frames
- safety goggles
- sunglasses with lenses popped out
- readers with lenses popped out

### (using glasses)



### x2 ties - can be anything for fastening:

- zip ties
- string
- elastic bands
- wire twist ties
- paper clips
- tape

### x1 sheet of clear plastic

- transparency film
- page protector
- cut 2-liter soda bottle

hole punch OR knife OR scissors

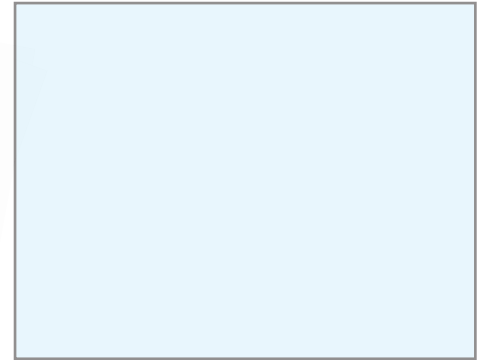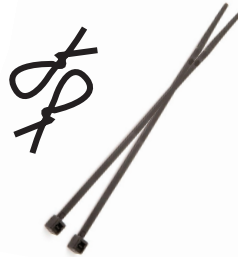


## INSTRUCTIONS

### 1. CUT 6 HOLES or SLITS in plastic sheet.


Cut Three slits or holes on each side, towards the middle-top, close to the outer edges:

- Two parallel horizontal slits for ties
- One vertical slit to thread the arms of the glasses through

Note holes are FARTHER APART than the width of the glasses.

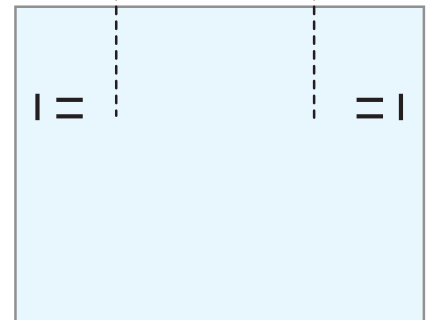

### 2. LOOP TIES LOOSELY through parallel holes.

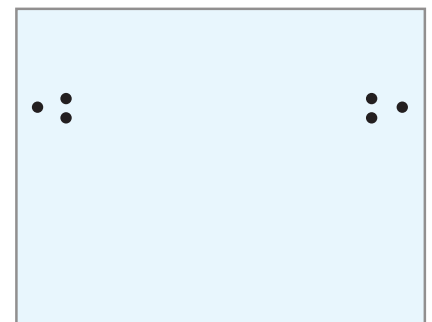

### 3. ATTACH to sides of glasses frame

Slide the arms of your glasses through the ties as well as the outermost/vertical holes/slits.

Glasses are worn INSIDE the shield.

Fasten/tighten ties around the arms of the glasses.

Attach towards the front of the face, allowing the plastic to curve forward and provide a gap of space between the shield and the nose/face.

For improved comfort, tie or twist on OUTSIDE of shield, not inside where it could poke+scratch the wearer's skin.

